amiea

EVOLUTIONLINE

MICROPIGMENTATION COLOR

Congratulations for your perfect choice!

Evolution Line permanent makeup colors are developed to achieve three major objectives: vibrancy, long-term color retention and safety. Safety comes first. For that reason, our pigmentation colors are developed in close collaboration with chemists and professional users.

We implemented features of highest standard:

- Sterilized by gamma radiation
- No preservatives added
- All pigments are tested in an external accredited analytical laboratory
- Secured with a seal of genuineness
- Meets and exceeds the requirements of ResAP 2008(1)
- MT.DERM GmbH is certified in accordance with ISO 13485

These features will ensure you the lasting experience of beautiful and safe permanent makeup. Detailed information about chemical composition provided in material safety data sheets

IMPORTANT NOTICE!

The permanent makeup procedure should be performed rather shallow. Avoid too deep penetration of the skin. Prefer to do some more work on the retouch rather than penetrating too deep. Deep penetration of eyebrow colors will cause grey/blue/black and migrated lines. Deep penetration of lip colors will cause lilac/blue/grey results. All amiea colors can be mixed to create the optimal color, EXCEPT of eyebrow/eyeliner colors with lip colors.

This recommendation was developed after intensive in-house testing.

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EN

Evolution Line pigmentation colors

Evolution Line pigmentation colors fulfil the highest safety and quality requirements. The colors are sterile and available in plastic bottles of 5 and 10 ml.

Please read the following important information before use:

Ingredients

aqua, isopropyl alcohol, glycerine, INCI No. (+/-): 77491, 77492, 77288, 77891, 77266, 77499, 56300, polyethylene glycol 8, propylene glycol, VP/VA Copolymer

- The detailed composition of the respective pigmentation colors is indicated on the product label.
- Pigmentation colors may contain traces of nickel and chromium.
- Ingredients may cause allergic reactions.

Intended use

The color pigments are embedded into the dermis with the aid of a micropigmentation needle and a micropigmentation device.

- Eyebrows
- Eye liner
- Lip shading or lip contours
- Tattoos

Contraindications **▲ WARNING**

- Haemophilia or other blood clotting disorders
- Systemic infection or acute local skin infections (as hepatitis type A, B, C, D, E, F or HIV infection)

- Uncontrolled diabetes
- All types of scars in the treatment area
- Do not use for treatment of areas of skin with dermatosis, e.g. skin tumors, keloids or in case of extreme predisposition to keloids, solar keratosis, warts, or birthmarks
- Allergic reactions to one or more ingredients of the pigment color(s)
- Allergy to topical anesthetics (local anesthetic)
- In case of infections accompanied by fever
- Acute inflammation of the conjunctiva (contraindicated when pigmenting eye-liner)
- Clients with a herpes simplex virus infection that have received no prophylactic antiviral therapy before treatment
- Clients on courses of high dose of chemotherapy, radiotherapy, or high-dosage corticosteroid therapy
- Pregnancy and nursing mothers
- Clients who are suffering from eczemas, exanthema or open wounds in the treatment area
- Clients who have undergone plastic surgery in the treatment area within the last year.
- Clients who have undergone filler injections or laser IPL treatments in the treatment area within the last six months
- Client is under the influence of alcohol and/or drugs
- Treatment on mucous membranes is strictly prohibited
- Treatment on eyeballs is strictly prohibited

If there is the slightest doubt about safety for the customer, the treatment must not be started or the treatment must be stopped immediately and a physician's release have to be obtained.

The treatment must be stopped immediately if client experiences:

- Excessive pain
- Excess secretion of wound fluids
- Fainting spells / dizziness

Use of the pigmentation colors **⚠ CAUTION**

- May only be used by medical professionals and by suitably qualified micropigmentation professionals
- Only use for the intended purpose
- **Shake well before use**
- Do not mix with products from other manufacturers
- Carry out a compatibility test prior to use *)
- Avoid contact with the eyes. In case of eye contact, flush with water
- Inform the customer about possible risks
- Recommend use with a hygiene module, since only this guarantees hygienic results
- All country specific hygiene requirements must be taken into account
- Ensure that the pigment color left in the bottle does not become contaminated during treatment
- Do not return color to bottle once removed
- Do not expose bottle to direct sunlight for prolonged periods
- Only clean bottle with a cloth soaked in disinfectant
- Do not drink or inhale color
- Use before the expiry date. The expiry date can be found on the product label (exp: month/year)
- If it must be assumed that the colors are not in their proper condition (i.e. transport damage, improper storage, etc.), the colors must not be used.
- Store in a cool and dark place at all times

- Use the color up within 6 months after opening the bottle
- Keep out of reach of children
- Leftovers can be disposed of with domestic waste

**) For the compatibility test, apply a small amount of the permanent make up color into the sensitive skin behind the ear by using a sterile needle. Wait for 72 hours to ensure that no allergic reactions occur. The same test should then be performed on the area of skin to be treated. Carry out this test for every pigment color which is to be used.*

Risks **⚠ CAUTION**

- The color that is used remains in the skin. It may not be possible to remove it again.
- Even though the utilised pigments are very stable, the color may fade or change after use.
- It is not guaranteed that the color will fade completely.
- Depending on application by the qualified professionals and the state of the skin, the color pigments may become unevenly embedded.
- Even though there are no known side effects thus far, undesirable allergic reactions may occur.
- It may not be possible to remove pigment color from clothing (or other materials).

Important information **⚠ CAUTION**

Despite continuous efforts to ensure that the product is not toxic and fulfils the highest requirements with regard to quality and purity, use of this product may trigger allergic reactions in some persons due to individual sensitivity towards the color.

⚠ WARNING

MT.DERM assumes no responsibility for allergic reactions of individual persons to colors, neither due to an assurance of general suitability for use, practicality or otherwise. If the tolerability of the color is not known from earlier applications, a pigmentation test must be done prior to use or a dermatologist consulted.

MT.DERM assumes no liability for consequences of attempts to remove the colors. In particular, the manufacturer shall not be liable in case of complications from removing the color pigments with thermal, mechanical, or other procedures (e.g. with laser treatment).

In case of complaints, please state the lot number on the product label in all cases.

MT.DERM quality requirements

- MT.DERM is certified in accordance with ISO 13485.
- Each pigmentation color has its own product file, which can be verified at any time by representatives of the US government and European authorities.
- The ingredients of the pigmentation colors correspond to the requirements of European Resolution ResAP(2008)1 for tattoos and permanent make-up.
- Microbiological harmlessness of the colors has been confirmed.

Warning symbols and statements used in this Instruction for Use

⚠ WARNING	WARNING indicates a hazardous situation which, if not avoided, could result in death or serious injury
⚠ CAUTION	CAUTION indicates a hazardous situation which, if not avoided, could result in minor or moderate injury

Definition of symbols used on the product and its packaging

The following symbols were used on the product label and the packaging for identification:

	Name and address of the manufacturer
	Please follow the operating instructions
REF	Item number of the product
LOT	Batch code of the product
	Use by
	Usage time after opening
STERILIZED	Sterilized

Manufacturer

MT.DERM GmbH
Gustav-Krone-Str. 3,
14167 Berlin, Germany
www.mtderm.com, info@mtderm.com
Evolution Line Pigmentation Colors Package Leaflet
© 07.2018

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